



MetriCide™ OPA Plus Solution

High-level Disinfectant for Semi-Critical Medical Devices (0.6% *ortho*-Phthalaldehyde Solution)

Reusable *ortho*-Phthalaldehyde (OPA) based high-level disinfectant (HLD) provides a broad spectrum of kill including Tuberculosis, Hepatitis A (AER), Hepatitis B, HIV-1, and Poliovirus Type 1.

- Gentle on flexible endoscopes and other surgical instruments
- Disinfects in 5 minutes at 25°C in an automatic endoscope reprocessor (AER) and may be reused for up to 14 days (when monitored according to label instructions for use)*
- Disinfects in 12 minutes at 20°C in manual reprocessing and may be reused for up to 30 days (when monitored according to label instructions for use)*
- 2 year shelf life and a 75-day open bottle shelf life
- Requires no activation or mixing

Part No.	Product Description	Packaging
10-6000	MetriCide OPA Plus Solution	4 bottles/case 1 gallon

* Test solution prior to each use to guard against dilution which may lower the concentration of OPA below its MRC.

Composition/Information On Ingredients

Component	CAS No.	Amount
Water and non-hazardous ingredients	Mixture	90-100%
Ortho- Phthalaldehyde	643-79-8	<1

Handling and Storage

Precautions for Safe Handling: Avoid contact with eyes, skin and clothing. Avoid breathing vapors. Do not form sprays, mists or aerosols of this product. Wear appropriate eye and skin protection when handling. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Remove and wash contaminated clothing before reuse.

Conditions for Safe Storage, Including any Incompatibilities: Store at room temperature.